

ReddyPort® Moisturizer Pack

Instructions for Use (IFU)

The applicator swab included in the Moisturizer Pack is used to moisten a patient's mouth.



Applicator and Suction Swab

The suction swab connects to wall suction and is used to apply the oral rinse, clean and suction excess fluid.



STEP 1: APPLY mouth moisturizer or water to swab.

WARNING: When applying mouth moisturizer to swab, **DO NOT** get gel onto protective guard. It can cause device to slide out of elbow.



STEP 2: RETRACT swab into protective guard to cover it. With swab covered, insert protective guard into elbow using finger grips.



STEP 3: INSERT protective guard into elbow using figure grips. Ensure protective guard is completely inserted.



STEP 4: APPLY mouth moisturizer to mouth and lips as needed.




STEP 5: REMOVE swab by gently pulling on the stick to retract into protective guard. Continue pulling until device slides out of elbow.

For drug facts information please refer to the Mouth Moisturizer packaging. **DO NOT** use product past the use by date.

For any inquiries contact ReddyPort at [1.801.899.3036](tel:18018993036) or email info@reddyport.com

To view our in-service video visit reddyport.com/resources

 SMD Manufacturing, LLC
918 S 500 W STE A
Salt Lake City, UT, 84101 USA
Manufactured for ReddyPort

reddyport.com

REDDYPORT