

# ReddyPort® NIV Maintenance

## Instructions for Use (IFU)


Color-coded packaging enables caregivers to track oral care compliance (Q4 protocols) easily by identifying specific component packs.

Prep Pack is color coded with **GRAY** packaging

Suction Toothbrush is color coded with **PURPLE** packaging

Suction Swab is color coded with **DARK BLUE** packaging


**CONNECT** suction handle to suction tubing. Suction tubing should be attached to a suction canister.

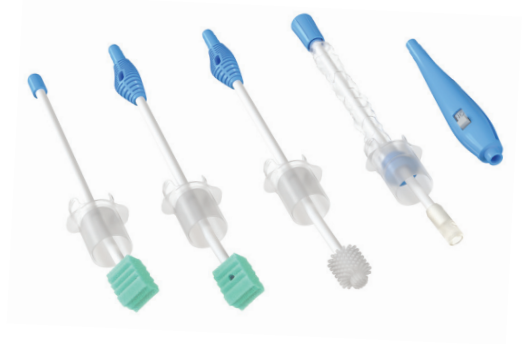


### Yankauer usage



**STEP 1: ATTACH** yankauer to handle. Keep at patient bedside.

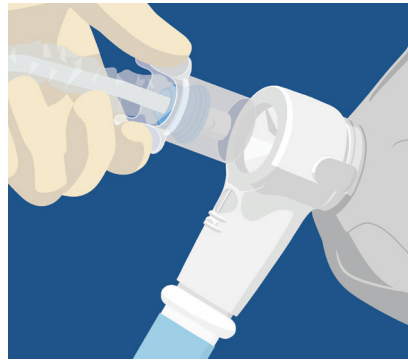

**STEP 2: INSERT** protective guard into ReddyPort® Elbow using finger grips. Gently slide suction tube through mask and into patient's mouth. Apply suction by toggling "on/off" slider on handle.

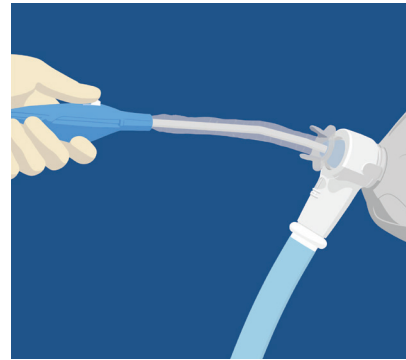

**STEP 3: REMOVE** yankauer by pulling gently on the handle until suction tip is recessed into protective guard. Continue until protective guard slides out of elbow.


**DO NOT** remove device using finger tips on protective guard.

### Suction swab and toothbrush

The suction swab and toothbrush connect to wall suction and are used to apply the oral rinse, clean and suction excess fluid.




**STEP 1: MOISTEN** swab or toothbrush by retracting protective guard and dipping in oral solution.

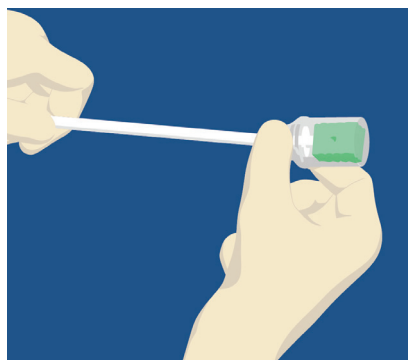


**STEP 2: RETRACT** swab or toothbrush into protective guard to cover it. With swab or toothbrush covered, insert protective guard into elbow using finger grips.



**STEP 3: INSERT** protective guard into elbow using figure grips. Ensure protective guard is completely inserted.

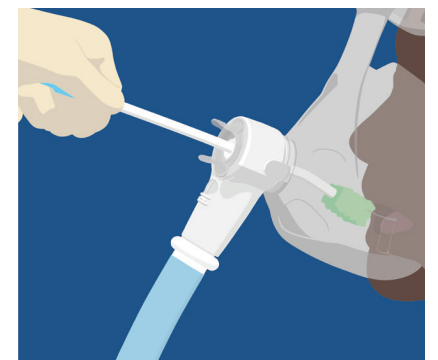

**STEP 4: ADVANCE** swab or toothbrush into patient's mouth gently to perform oral hygiene for 1 to 2 minutes. Suction excess fluid by placing a finger over the suction port.

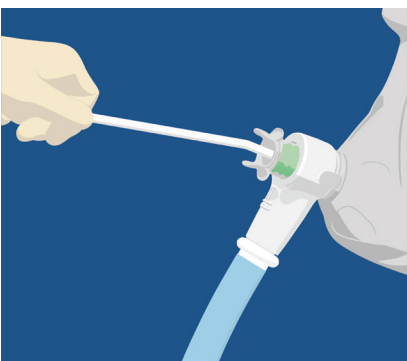


**STEP 5: REMOVE** swab or toothbrush by gently pulling on the stick to retract into protective guard. Continue pulling until device slides out of elbow.

#### Applicator Swab

The applicator swab included in both suction swab and toothbrush packs is used to moisten a patient's mouth with ice water or moisturizer gel.

- **APPLY** ice water or mouth moisturizer to swab.
- **FOLLOW** suction swab instructions (steps 2-4) from Suction swab and toothbrush section above.
- **INSERT** swab into the elbow.
- **APPLY** mouth moisturizer to mouth and lips as needed.
- **REMOVE** the applicator swab (step 5).

**DO NOT** remove device using finger tips on protective guard.

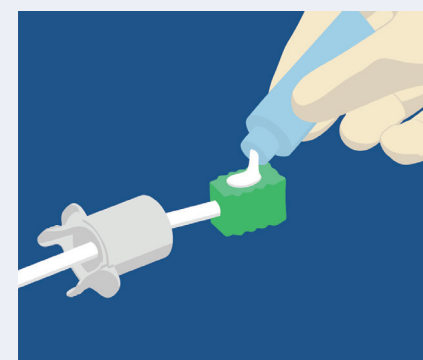


**WARNING:** When applying mouth moisturizer to swab, **DO NOT** get gel onto protective guard. It can cause device to slide out of elbow.

ReddyPort Maintenance products follow Q4 protocols (every 4 hours) for standard oral care procedures.

For drug facts information please refer to the ReddyPort Maintenance packaging.

For any inquiries contact ReddyPort at **1.801.899.3036** or email [info@reddyport.com](mailto:info@reddyport.com)

To view our in-service video visit [reddyport.com/resources](http://reddyport.com/resources)

 SMD Manufacturing, LLC  
918 S 500 W STE A  
Salt Lake City, UT, 84101 USA

Manufactured for ReddyPort

For Symbol Glossary visit [smdmanufacturing.com/symbolsglossary](http://smdmanufacturing.com/symbolsglossary)

[reddyport.com](http://reddyport.com)

REDDYPORT